

INSTRUCTION FOR COVID-19 SURFACE TESTING

YOUR KIT CONTAINS




Instruction



1 Swabs in
buffer vials



1 Pair of
Gloves

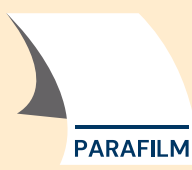

Parafilm
Tape



1 Package
Wipes



1 Plastic Bag



Chain of
Custody
Form



Return
Envelope

BEFORE YOU BEGIN TESTING

1

CHECK THE CONTENTS OF YOUR KIT

The number of swabs you receive should be the number of swabs you ordered. Contact Purity Laboratories (Purity) if you are missing components at info@puritylabsinc.com.

2

READ ALL INSTRUCTION

Before you begin, read through all the instructions. If you have questions, please contact Purity Laboratories at info@puritylabsinc.com before starting the test.

3

LABEL EACH SWAB CONTAINER WITH A UNIQUE ID

Use indelible, waterproof pen or marker for labeling. You may want to identify the object of the test E.g. "conference table", "reception desk", "front door", etc.

4

PUT ON YOUR MASK

(Not provided) Use triple layer disposal mask or a personal mask that will provide good protection.



5

PUT ON YOUR GLOVES

(Provided in your kit).




PERFORM SWABBING

6

HANDLING THE SWAB AND BUFFERING SOLUTION

1. Unscrew the swab from the buffering solution vial bottle.
2. Dip the swab into the buffering solution to moisten the tip.
3. Swab an area approximately 5 cm by 5 cm (approximately 2 in by 2 in) by swabbing in one direction horizontally, then over the same area, swab in one direction vertically.
4. Place the swab back in the buffering solution with the tip of the swab immersed in the buffering solution.
5. Form a tight closure for the buffering solution vial by screwing the swab into the vial.
6. Wrap parafilm around the closure for extra protection.

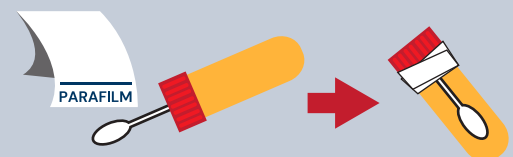

INSTRUCTION FOR COVID-19 SURFACE TESTING

7 PLACE BUFFER VIAL WITH SWAB IN PASTIC BAG

Place all buffer vials (with swabs) into the plastic bag provided. Wipe the outside of the bag with the disinfectant wipe provided.



8 COMPLETE ORDER CHAIN OF CUSTODY FORM

Read the instructions on the Chain of Custody sheet provided in your kit. Complete the form.




9 PACKAGE UP YOUR TEST TO SEND TO THE LAB

Reuse your kit box to send your test back to the lab.

You must include these items:

1. The swabs labeled and tightened in the buffering solution vials.
 - (Check to see if closure is tightened).
2. A completed Chain of Custody form.

10 Place the kit in the Fedex envelope addressed to Purity, which is provided in your kit.

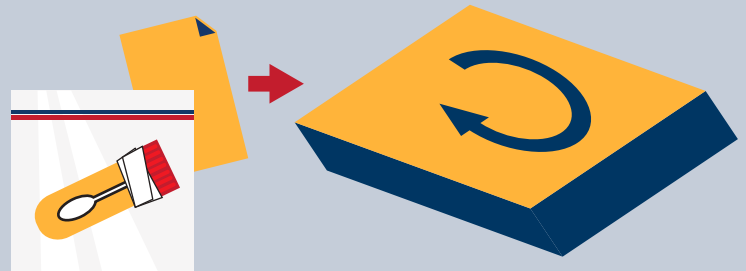

SEND TEST TO LAB - GET RESULT

11 SEND TO PURITY LABORATORIES

Drop off the package at their local FedEx center.



12 GET YOUR RESULT BY EMAIL

